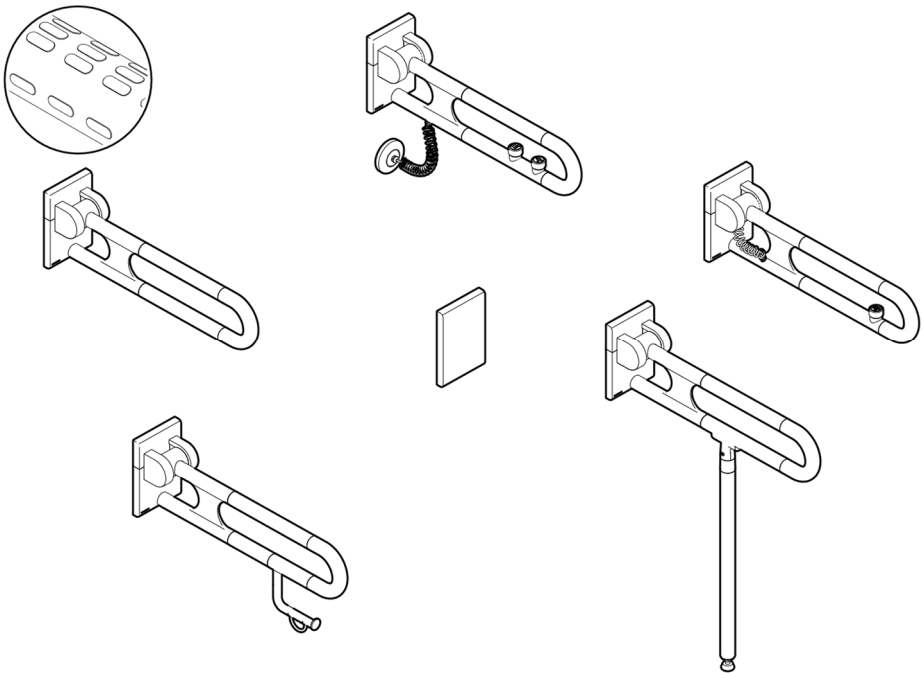
# NORMBAU

PASSION FOR CARE

## Nylon Care 300/400

- (DE) Montage- und  
Gebrauchsanleitung
- (EN) Fixing instruction and  
directions for use
- (FR) Instructions de montage et  
d'utilisation
- (NL) Montage- en  
gebruiksaanwijzing

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### **Intended use**

NORMBAU products are designed for use in wet rooms, at home, in care facilities and hospitals. They provide grip and support and assist the independent standing and sitting of persons with or without physical impairment.

### **Disclaimer of liability**

NORMBAU assumes no liability for personal injury or property damage that arises or has arisen as a result of improper assembly, use or cleaning, in particular in disregard of the warning and safety instructions listed in these instructions. According to the current regulation, the warranty is two years.

Under the intended conditions of use and in compliance with the guidelines for maintenance and cleaning, the intended lifespan of the article is five years.

### **Warning notices**

#### **Risk of injury**

If the max. loading permitted is exceeded, it can lead to permanent damage to the product. In this case, the product must be tested before further use!

#### **Risk of injury**

When folding down the seat, ensure that there is no part of the body under the product.

There is no toxic hazard from the material used by NORMBAU. Tested according to DIN EN ISO 10993-5:2009. Product surfaces can become very hot when exposed to external heat sources.

### **Safety instructions**

Installation is to be carried out by a specialist and only with NORMBAU mounting sets or suitable fastening material. Before installation, the wall structure must be checked to determine the fastening material. Observe the information on the max. loading permitted and the instructions for mounting! Following installation, the instructions must be made available to the user.

In case of questions, mor information, uncertainties or if damage occurs, contact the NORMBAU support service +49 (0) 7843 704 0 or visit us at [www.normbau-extranet.de](http://www.normbau-extranet.de).

### **Load**

Max. loading permitted of listed products: 100kg

0300 xxx    0437 xxx    4300 xxx    4437 xxx

Max. loading permitted of listed products: 125kg

0447 xxx    4447 xxx

With max. length and max. loading permitted, the max. force per fastening is 3kN.

### **Maintenance**

NORMBAU products are note maintenance-free! The products are to be tested monthly by qualified specialists for proper function. In the event of defects (e.g. insufficient braking force), the assembly instructions must be observed.

### **Care and cleaning instructions**

#### **Risk of injury**

Improper cleaning can cause damage to the surface and lead to injuries.

Nylon products from NORMBAU have a smooth, high-gloss, easy-to-clean surface. Dirty parts can be cleaned with a soft, damp cloth or sponge. If heavily soiled, we recommend a commercially available detergent. Common disinfectants can be used for wipe disinfection. Acidic cleaning agents, care agents, disinfectants and decalcifying agents must always be used in accordance with the instructions for use. An acid content of over 10% can attack the surface and negatively change the brilliance and color. Use all agents only diluted, leave to act briefly and rinse off sufficiently with water. Abrasive, strongly acidic and alcoholic cleaning and care products should be avoided with Nylon.

### **Disposal**

Disposal of the product and packaging is subject to the relevant national regulations and must be disposed of in the prescribed manner.

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EN

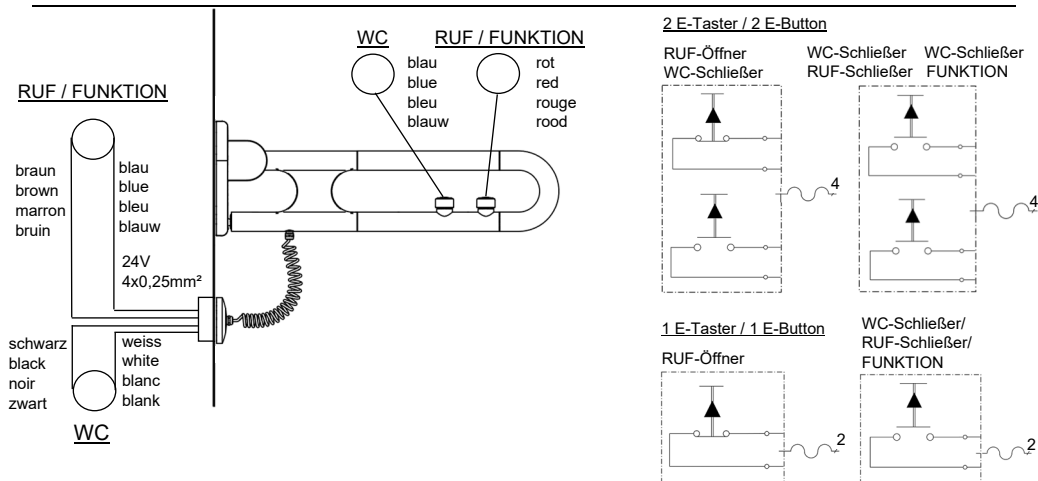
On walls for masonry:  
 62mm dia. recess for flush-mounted socket  
 On lightweight walls:  
 milled hole diameter 68mm dia. for cavity wall socket

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EN

If the e-switch is connected as a call button on an existing intercom according to VDE 0834, a cable relief must be fitted in the junction box. E-Button standard configuration “normally open contact”, also available in configuration “normally closed contact”. A minimum of protection type IP X4 must be provided for the junction box (junction box Up6). The permitted maximum voltage for WC controls or for emergency call systems is 24V.

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NORMBAU GmbH Schwarzwaldstr. 15 77871 Renchen   DE
**NORMBAU GmbH**